

Local

T: (07) 3808 6833

F: (07) 3808 6108

E: mail@spectronicsnz.com

A.B.N. 15 011 046 585

International

T: +61 7 3808 6833

F: +61 7 3808 6108

W: www.spectronicsnz.com

Inclusive Learning Technologies #17 LTD

PO BOX 88

Rosedale

QLD 4123

AUSTRALIA



Backing up Proloquo2Go to a local computer

You can now backup all your Proloquo2Go vocabulary changes to any computer which is on the same network as your iPod Touch or iPhone which contains Proloquo2Go.

Your iPod will need to be connected to the network wirelessly. Your computer can be connected wirelessly or through an Ethernet cable. Your computer will also need to have Firefox or Safari installed as a web browser. Internet Explorer is not compatible with this process. To download Firefox go to www.firefox.com. To download Safari go to www.apple.com/downloads.

To complete the backup, please follow these steps:

- 1) Connect the iPhone/iPod Touch through Wifi.
- 2) Make sure your computer also is connected to the network through WiFi or Ethernet.
- 3) On the iPod Touch/iPhone, launch Proloquo2Go and go to the Options menu
- 4) Select the "Connect to Computer" option
- 5) Follow the instructions on the iPod Touch/iPhone screen until you see the instructions to launch Firefox or Safari and the address that you need to type into the address bar of the browser.
- 6) On your computer, launch your web browser (either Safari or Firefox). In the address bar of the web browser, type the address displayed on the iPod Touch/iPhone screen which begins with <http://>..
- 7) On the page that appears on your screen, the top half is designed to help you restore Proloquo2Go from a backup stored on your computer and the bottom half lists all the different Proloquo2Go backups stored on your iPod Touch/iPhone. Select the backup you would like to save on this computer and choose where on your computer you would like to save it to.

Please note: This process can be done on any network with WiFi that both an iPod Touch/iPhone can connect to. This process is completely independant of iTunes – so you can backup at school or at home and don't need to worry about limiting it to a computer that the iPod syncs with. If Proloquo2Go is accidentally deleted from the device you will still need to Sync the device to reinstall it – but the backup can then be restored from wherever it has been performed.

If you have any questions, please contact Spectronics on (07) 3808 6833 or (03) 9848 4812.